

Disinfection

hup DES FORTE M

Alcohol-based surface disinfectant wipes


Product name

hup DES FORTE M

Product range

Disinfection

Product description

Ready-to-use, alcohol-based, fast-acting disinfectant wipes for all alcohol-resistant surfaces. Very good material compatibility and skin tolerability. VAH/ÖGHMP listed. Perfume-free. For rapid disinfection also during the working process such as in kitchens, gyms, residential care homes, and doctors' surgeries.

Packaging unit

100 towels / pack

Art.-No. 4450304542

Field of application | use

Biocides Ordinance: Ready-to-use alcohol-based wipes for quick disinfection of alcohol-resistant surfaces.

EU Medical Devices Directive: Disinfection of non-invasive medical devices by professionally trained personnel.

Application: Use the hup DES FORTE wipes to fully wet the surface and allow the solution to take effect. Make sure that the surface is fully covered and stays moist the whole time. All visible contaminations must be removed before disinfection.


UDI-DI

00: UDI-DI 09008353219842

55: UDI-DI 09008353219910

Activity spectrum

Surface disinfection with mechanical action: bactericidal, yeasticidal	VAH/ÖGHMP; EN 13624, EN 13727, EN 16615	1 min *
Limited virucidal	EN 14476	1 min *
Limited virucidal PLUS	EN 14476	3 min **
Adenovirus	EN 14476	3 min **
Norovirus	EN 14476	1 min **
Rotavirus	EN 14476	30 sec **
Mycobactericidal	EN 14348	1 min **
Limited virucidal	EN 16777	1 min *

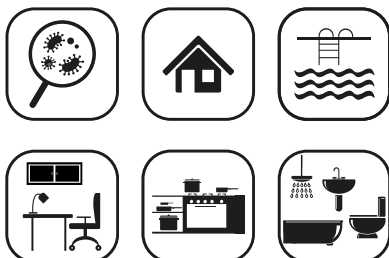



Disinfection

hup DES FORTE M [Medizinprodukt]

Alcohol-based surface disinfectant wipes

Limited virucidal PLUS	EN 16777	5 min **
Bactericidal/yeastocidal	EN 1276, EN 1650, EN 13697	1 min *

* Dirty conditions, medical field. ** Clean conditions, medical field.

All tests performed at room temperature.

Material compatibility

Applying the solution onto perspex (PMMA) is not recommended.

Dimensions

H x W: 20 x 20 cm

Safety information

Please follow the standard safety measures when handling chemicals. **Use disinfectant safely. Always read the product label before use.** Keep out of reach of children. Keep container tightly closed. Keep away from sources of ignition – no smoking. Only use visually perfect goods.

Storage

To maintain product quality, store in a cool, dry place.

Product recommendation

Ideally suited for use with the XIBU 2WIPE hybrid - the world's first automatic dispenser for wet wipes. The patented dispenser protects the wipes from drying out and germs. We recommend hup DES PLUS for surfaces sensitive to alcohol.

Selling unit

Package

Euro pallet

270 packages

Rev. No. 02 / 06.05.2024